Quick start guide

This concise guide will help you get started with your Birth Alarm Mobile Premium/Lite and it includes brief references to the comprehensive manual that can be downloaded from our website <http://www.birthalarm.com>. These references take the following format: “(see 2.3)”

First use (perform 1x after purchase):

1. Unscrew the Birth Alarm Mobile Premium/Lite and remove the red label from the pole of the battery. The batteries are now in contact with the Birth Alarm Mobile Premium/Lite.
2. Depending on your country of residence, the SIM card may have been provided.
 - If this is the case, then the SIM card will already be inserted into the SIM holder beneath the battery compartment. (see 5.3)
 - If this is not the case, then you must purchase and set up the SIM yourself (see 6). When you have done so, the SIM card must be activated and slotted into the SIM holder. (see 5.3)

Once you have performed these steps on your new Birth Alarm Mobile Premium/Lite, you can reseal it with the screws and proceed with the set-up.

Get started quickly

Programming:

1. Take the Birth Alarm Mobile Premium/Lite out of the case and take hold of the special screwdriver.
2. Hold the rear end of the screwdriver against the red square next to the  symbol for a few seconds. (see 3.2)

3. Whilst you are doing this, the status LED will first flash green 3 times, then blue. When the status LED turns blue, you can remove the screwdriver. *(The Birth Alarm Mobile Premium/Lite is now in programming mode) (see 7.1)*
4. The Birth Alarm Mobile Premium/Lite will now connect to the provider. *(if you have installed a new SIM card, the Birth Alarm Mobile Premium/Lite may perform a number of one-time configurations. This may take several minutes)*
5. When the Birth Alarm Mobile Premium/Lite has established a connection to the provider, the status LED will flash 1x blue and 1x green alternatingly.
6. Now call the Birth Alarm Mobile Premium/Lite with your own (preferably mobile) telephone. The number for the Birth Alarm Mobile Premium/Lite is stated on the card or on the sticker from the prepaid package that you purchased or which was supplied with the Birth Alarm Mobile Premium/Lite. *(see 7.2.1) (Note! Your mobile phone's caller display feature must be switched on. This means that when you call someone, that person must be able to view your mobile number on the screen of their telephone. If necessary, consult the user manual for your mobile telephone)*
7. The Birth Alarm Mobile Premium/Lite will receive your call and will store your number in the memory.
8. The Birth Alarm Mobile Premium/Lite will now call you back to confirm the number. You do **not** need to answer!
9. The Birth Alarm Mobile Premium/Lite is now programmed for use! After a few seconds, the status LED will alternate between flashing 1x blue and 2x green. *(You now have the option of programming a 2nd, 3rd, and 4th number using the same method. There are 2 alternative methods for programming the internal SIM card. These are by SMS or direct programming on another mobile telephone.)*
10. When you have finished programming the device, wait approximately 2 minutes until the Birth Alarm Mobile Premium/Lite automatically leaves

programming mode and returns to normal status (*When the status is normal, the status LED will flash once briefly every 8 seconds*).

Testing:

1. Switch on the Birth Alarm Mobile Premium/Lite (*see 10*) and then turn the Birth Alarm Mobile Premium/Lite until it is in a vertical position.
2. Wait several seconds until the GSM LED starts flashing (*if you have a Birth Alarm Mobile Premium and you use the sleep mode, then this will only activate when the device is returned to a horizontal position after 20 seconds*).
3. The Birth Alarm Mobile Premium/Lite will call you in a moment. Allow your (mobile) telephone to ring out.

Installation:

1. Always check that the Birth Alarm Mobile Premium/Lite is working prior to use. (*see 10*)
2. Switch on the Birth Alarm Mobile Premium/Lite. (*see 3.2*)
3. Slide the Birth Alarm Mobile Premium/Lite into the holder provided and close this using the clip (*see 11*) (You can easily switch the Birth Alarm Mobile Premium/Lite on and off while it is in the holder).
4. Now install the holder under the halter, or, if desired, into the ring on an anti-roll girth using the cable ties (tie wraps).
5. If you are installing the Birth Alarm Mobile Premium/Lite under your mare's halter, it is important that the holder is secured reasonably tightly under the jaw of the mare. (*When the mare is holding its head upright, Birth Alarm Mobile Premium/Lite should hang in a fairly horizontal position*)

Maintenance:

1. Check that the battery level is sufficient occasionally when visiting your mare. You can do this quickly by checking the colour of the status LED. This should flash green once every eight seconds. If you see a red flash

also, you need to replace the batteries. *(Always replace all 3 batteries simultaneously. Rechargeable batteries should not be used!)*

2. Check the credit on the SIM card occasionally. In order to do this, remove the SIM card from the BA Mobile Premium/Lite and insert it into your own mobile telephone. Next, call the credit/top-up service line for your provider and follow the instructions.

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